

EZ Ride+™

INSTRUCTIONS



**READ
THIS
BEFORE
USING**

EZ Ride+
PATENT PENDING

Read Instruction Manual Completely Before Using EZ Ride

**Multi-tool
Included**



EZ Ride+
PATENT PENDING

More info at: www.ezrideplus.com

Copyright 2021 Shield Innovations LLC



WARNING!

Carefully read this manual regarding EZ Ride. EZ Ride is not a toy. Use caution when attached to your wheel chair.

THE MANUFACTURER DISCALIMS ALL LIABILITY RELATING TO DAMAGE CAUSED BY:

- The failure to abide by the instructions in this manual and its instructions.
- EZ Ride being used in ways which is not the intended use.
- EZ Ride being used in ways contrary to applicable laws on safety.
- Tampering with parts that operate the EZ Ride in an efficient manner.
- Incorrect assembly.
- Changes made to EZ Ride function and design without permission of the manufacturer.
- Use of non-original parts or parts that are not suitable for the EZ Ride.
- Repairs not performed by a qualified technician.

WARNING

MAX SPEED: 12mph
WEIGHT LIMIT: 300lbs
FOR WHEELCHAIR USE ONLY. NOT A TOY.

CAUTION

NO SHARP TURNS AT MEDIUM OR HIGH SPEEDS!



NOT FOR USE IN SAND

LOW TO MEDIUM SPEED IN ROUGH TERRAIN



BATTERY MUST NOT BE CLOSE TO EXTREME HEAT OR COLD.

MAX 140°F
MIN -4°F

Patent Pending



WARNING


LI-ION 36V === 350 Wh 10.0 Ah
Battery / Pile / Bateria 130501002DG9

To reduce the risk of injury, user must read Operator's Manual. Risk of fire and burns. Do not expose battery to Rain, Water, or Liquids, Open, Crush, Heat Above 140°F, Incinerate or Short Terminals. Follow Manufacturer's Instructions. Use only with charger listed in Operators Manual.

Remove Battery from unit when storing for long periods of time, changing attachments, or making adjustments.

Shield Innovations LLC
107 Pinckney St.
Oldsmar, FL 34677
888-987-4769


Tow Bar Parts

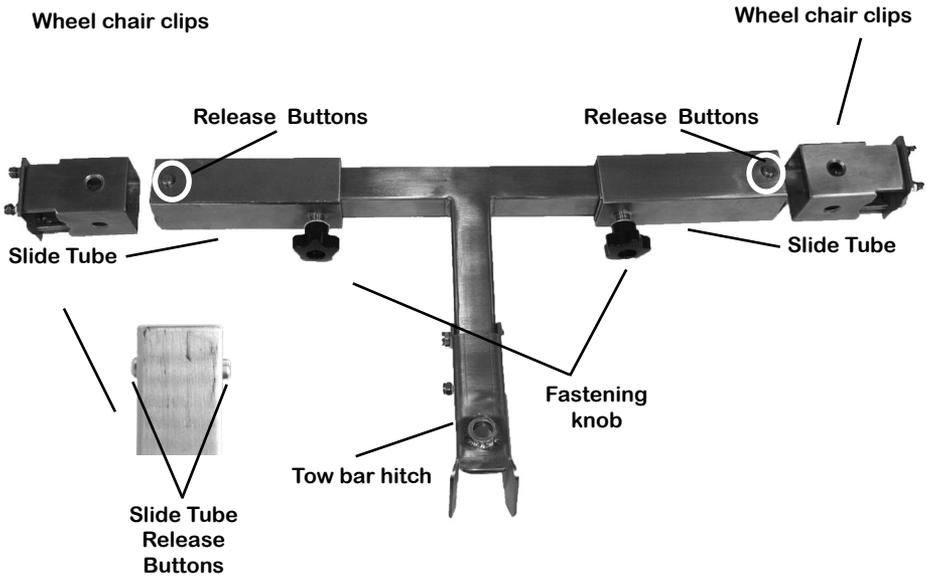

Chair Clip Breakdown



Wheel chair clips

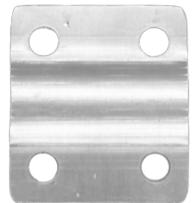

Chair clip cover

Hitch Breakdown



Receiver



Ajstment Screws



Tow Bar Installation:

Chair clip




Place chair clip around horizontal or vertical wheel chair bar. Close to front of chair with screws facing outward.



Place chair clip cover over bar. Confirm it matches the curvature of the bar.



Screw small nut first until it meets the cover, do not tighten. Then place second lock nut on screw. **Do not tighten** completely.

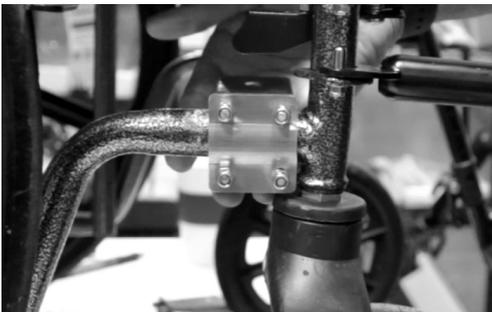


Repeat on other side of chair. Make sure left and right clip are in the same position on the bar. Adjust clips to make as level as possible

Tow Bar Installation:

Slide Tube



Make sure the chair is in the expanded position. The tow bar is spring loaded. Compress the slider tubes and line up with the chair clips.

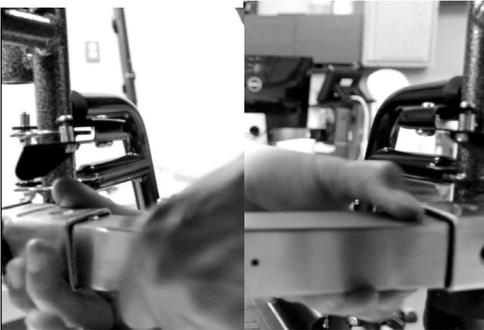

Squeeze both the top and the bottom release buttons and push the slide tube into the chair clip until the release buttons pop through the chair clip hole. Do on both sides.




Move the tow bar hitch until it is centered between the two foot pads. Then tighten the 2 chair clips making sure the tow bar is level.



Tighten the 2 fastening knobs. It is normal for the tow bar to have some slight movement.

Tow Bar UnInstall

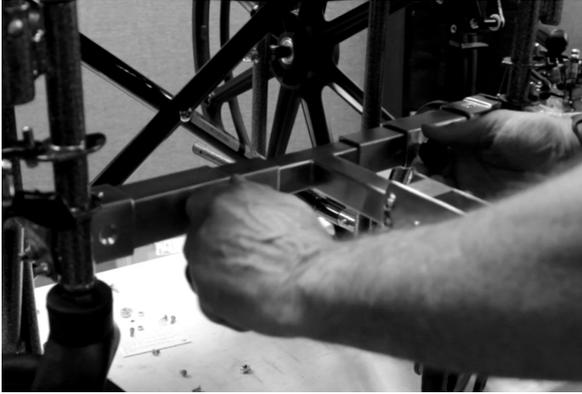

Loosen both fastening Knobs.



With one hand squeeze the chair clip release buttons and pull the slide tube toward the center until it releases from the chair clip.



Do the same on the other side. When both are released push both slide tubes to the center of the tow bar then remove from clips and tighten fastening knobs.

Tow Bar Hitch Height Adjustment




Normal position for EZ Ride tow bar hitch.



Raise tow bar hitch to raise front wheels of the chair higher off the ground.




Lower tow bar hitch to lower front wheels closer to the ground.

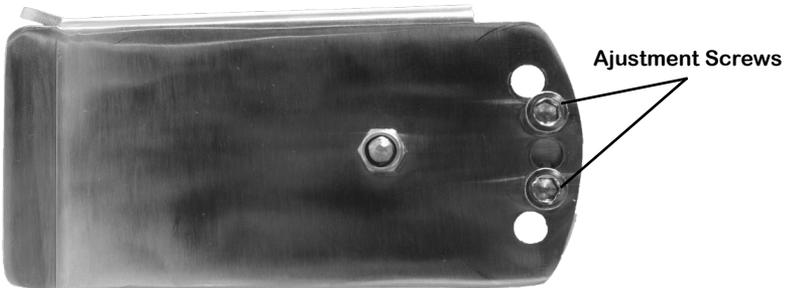


EZ Ride Kick Stand



To install kick stand insert kick stand over the plate with the two screws.

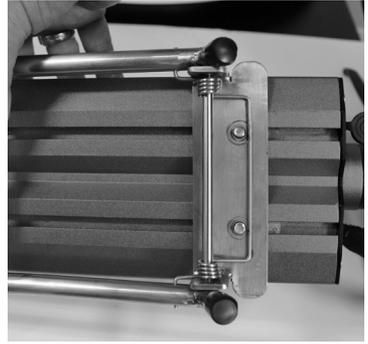


Make sure kick stand is straight along back side of EZ Ride.

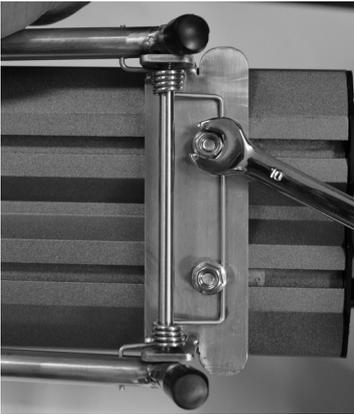


Install two screw nuts and tighten with wrench.


KickStand

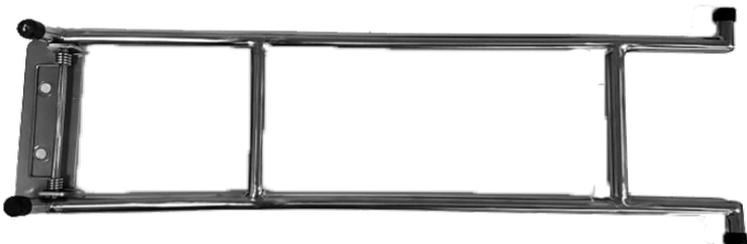

EZ Ride Chair Install

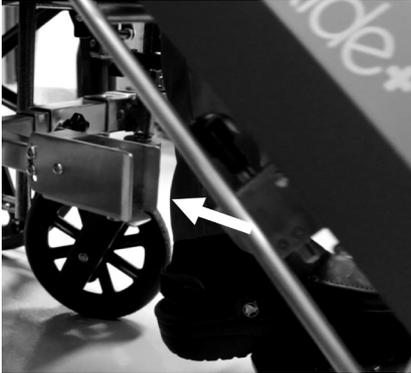


While sitting in chair, using steering handle pull EZ ride towards tow bar hitch. Kick-stand will automatically close.

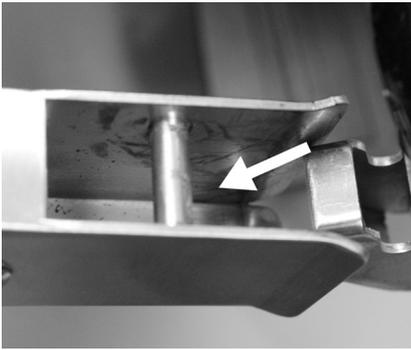


Line up EZ Ride handle inside hitch receiver so handle is inserted **under** receiver rod .

Receiver Rod

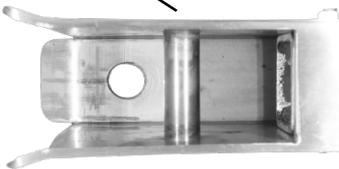

Tow Bar Hitch

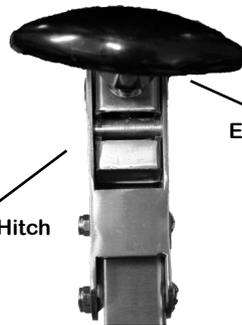


EZ Ride Handle



Pull EZ ride towards you and Once handle is under rod push up to engage tow bar hitch. Once locked in confirm front wheels are off the ground by at least 2-3". If not, adjust Towbar Hitch height. *see page 5*

EZ Ride Chair Release

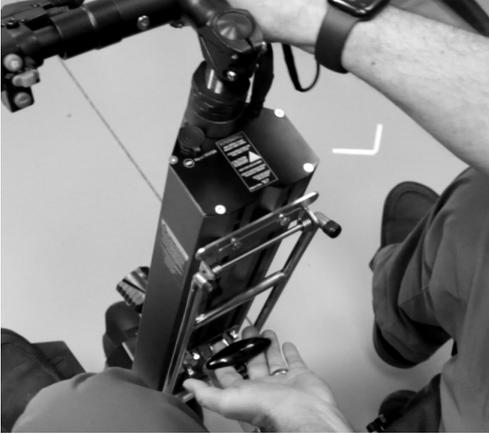

To release EZ ride from Tow Bar push up on steering handle and at the same time lift EZ Ride handle up.




Holding on to EZ ride handle and steering bar Push down and forward to release EZ Ride from Tow Bar Hitch. Push EZ ride away from Chair to clear footrests



Pull kick stand away from EZ ride to the extended position. Insure it is flush to the ground. Make sure Ez Ride is on level ground when extending Kick Stand

Speedometer & Handle Controls

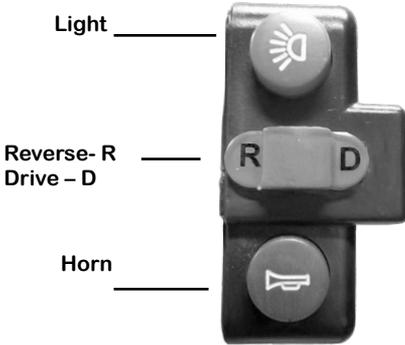

Display functions:
Speed display, Battery level, Total mileage, Cumulative mileage, current speed

Short presses to change speed

Press and hold to power on
Note: Some models will not have light function

Press to see mileage/
Voltage/Trip. When trip is showing press M button two times to record mileage



Throttle Controls

ECO Speed 3 mph- this gear is designed to allow user to cruise alongside a person walking as well as indoor use.
Note: Pressing throttle to start ECO speed has 3-second safety delay before starting.

MID Speed 6 mph- this gear is a moderate speed for outdoor use, do not make any sharp turns.

HIGH Speed 12 mph- CAUTION. This gear is fast. Use only on smooth surfaces in open conditions. **DO NOT MAKE ANY SHARP TURNS IN THIS SPEED.**

How to set cruise control.

- 1) Hold throttle on desired speed for 5 seconds. This will set the cruise control for that speed.
- 2) To release cruise control apply brake or push throttle.





EZRide Handle Fold



To fold handle to down position grab round release ring and pull towards handle grip.




Once release ring extended push handle to down position

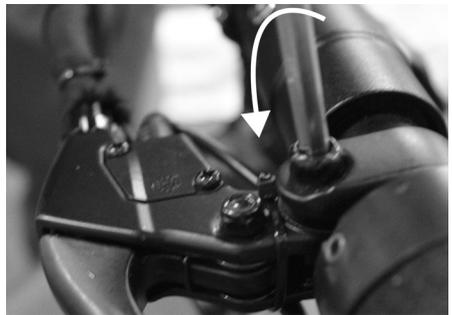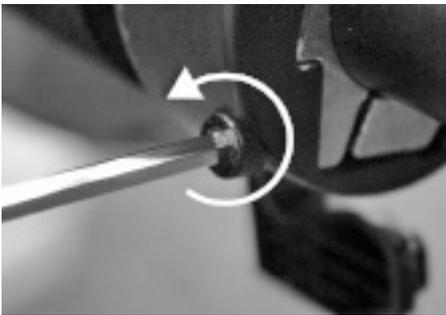

To adjust angle of Brake or Speedometer turn Allen screw counter clockwise to loosen. Adjust to position desired and tighten Allen screw

How To Charge

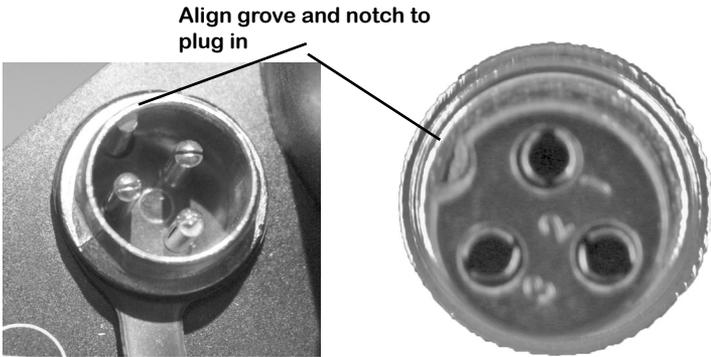

Insert three prong charging cable into port on top of EZ Ride.

Plug into normal 110 outlet

Keep plugged in until light on charger turns green.

Best to charge after each use.

Note: like a car EZ Ride will lose Charge over time if not used on a regular basis . Check battery before using EZ Ride

Battery Charger Travel Bag



Battery charger bag can be used for back of Wheel chair mounting or clips can be joined together for hanging or shoulder use.






READ THIS BEFORE USING


EZ Ride+[™]
PATENT PENDING

Read Instruction Manual Completely Before Using EZ Ride

-  **DO NOT use medium or high speed indoors. Use eco speed only.**
-  **DO NOT make a 360 degree turn using medium or high speed.**

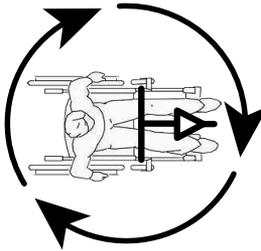

-  **DO NOT use EZ Ride on an incline of more than 30 degrees.**

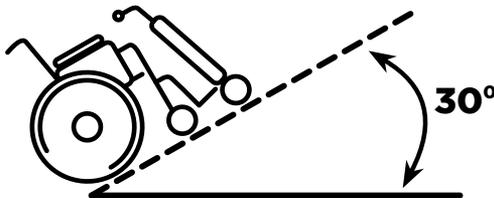

-  **DO NOT use high speed until you are comfortable with eco & medium speed.**
-  **DO NOT use EZ Ride in sand or loose gravel.**

EZ Ride+[™]
PATENT PENDING

More info at: www.ezrideplus.com